

Buty Oxy

Bubble Refine Facial



Contents

02
second part Instrument
1,Dynamic-oxygen skin technology
2,Rejuvenating RF Technology

01
first part Machine
1,Overview
2, working principle

03
treatment process

04
Use efficiency

05 Instrument advantages
Course settings

06
Instrument comparison
Applicable symptoms

07
Before and after comparison

08
common problem

First part Machine

1,Overview

Oxygeneo is a skin treatment instrument that combines radio frequency and dynamic oxygen perfectly. It is also the world's most advanced touch-type intelligent mobile oxygen therapeutic instrument. It can make collagen fibers shrink, regenerate and reorganize, and witness the automatic production of skin. The miracle of oxygen, in order to achieve superior efficacy of firming, lifting, whitening and blemish, is currently the world's leading anti-aging beauty skin project, providing instant and long-term visible results, providing customers with safe, effective, simple and fast Skin whitening, smoothing, wrinkle removal and enhanced skin anti-aging solutions.

2,working principle

1). Collagen Regeneration:

People refer to collagen as the "spring" and "reservoir" of the skin. In the dermis, collagen is as high as 70 to 80%. It is an adhesive between cells that supports the shape of the organ and also maintains cell activity. In the skin, collagen determines not only the elasticity and firmness of the skin, but also the moisture content and the water storage capacity of the skin. It is like a woven dense mesh that locks the skin tightly with nutrients and moisture, leaving the skin moist, smooth and elastic.


At a temperature of 37-41°C, the collagen fiber shrinks. After that, excess space is vacated. New collagen is generated and filled in. In this way, the collagen fills the entire space, making the tissue fibers plump. The skin will also be more compact. In addition to eliminating wrinkles, filling dents, or lifting the face, collagen can significantly improve skin texture and eliminate skin aging. The results are immediate and can instantly return the skin to a younger state. The spirit, enhancement, firmness and fullness can achieve a good effect of reversing the age of the skin.

2).Bohr effect

a. The unique deep cleansing oxygen head of oxygeneo instrument consists of sodium bicarbonate and citric acid. The gel contains natural and unique active ingredients. When deep cleansing oxygen heads and nutrient-rich gels come into contact with the skin surface, the chemical reaction between the two can generate large amounts of carbon dioxide. The frequency and amplitude of vibrations of the dynamic oxygen handles are specifically designed to generate power to maximize the production of chemistry. The reaction produces a large amount of carbon dioxide.

B. In a weak acid environment rich in carbon dioxide, the phenomenon of increased oxygen in the skin's tissue structure is called the Bohr effect. Through the rich carbon dioxide weak acidic environment and the infusion of active ingredients, the skin cells automatically generate oxygen and regenerate. The "Bohr effect" can: enhance blood circulation, increase oxygen content in blood, and accelerate skin cell metabolism

Compared with before and after treatment, whether it is the flatness of the skin, the compactness of the subcutaneous tissue, or the repair of the elastic fiber, the instant improvement has been achieved.

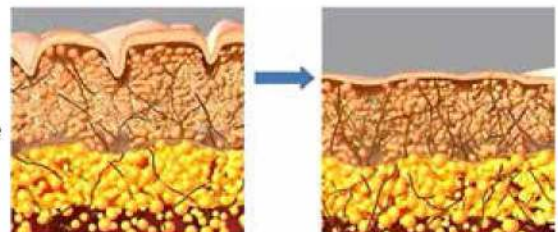


The second part Instrument

oxygen Dynamic technology

The oxygen skin handpiece is made up of three technologies, with two kinds of immediate effect products, and immediately complete the needs of wrinkle rejuvenation, blemish whitening, replenishment, skin rejuvenation, shrink pores.

Kinetic cleansing

Elimination of the skin's non-hydrophilic stratum corneum increases the permeability of the epidermis, opens up channels for deep nutrient absorption, and enhances nutrient penetration.

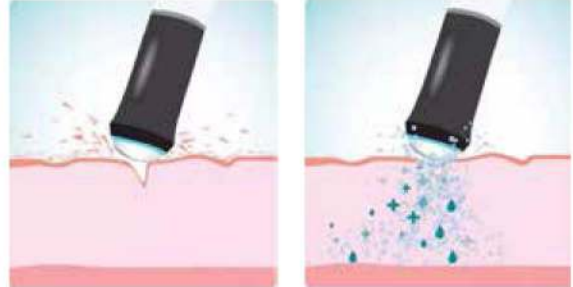

Oxygenation

Apply NeoBright efficacy products. When it touches the skin, millions of microscopic carbon dioxide bubbles are generated, awakening the skin's natural ability to supply oxygen and multiplying the efficacy of the product.



Import essence


After microdermabrasion of the epidermis to open the nutrient channel and enhance oxygen absorption and skin absorption, NeoBright's active substances such as hyaluronic acid, azelaic acid, protein peptides, and retinol are introduced into the skin. .



Rejuvenation RF Technology

The main characteristics of multi-polar radio frequency technology are low energy application, high energy density, deep level of effect, significant results, and high safety. The electrode size of the multi-pole RF electrode is perfect for the face, hand, and deep V.

- 1, 1MHz action frequency, safe and effective skin remodeling function;
- 2, high energy density, the first time that there is a clear effect;
- 3, clinically proven long-term efficacy is also very good;
- 4, the current is concentrated between the electrodes, does not require skin cooling;
- 5, 600mA low energy consumption, high comfort, good customer feedback;
- 6, suitable for a variety of skin and skin color;

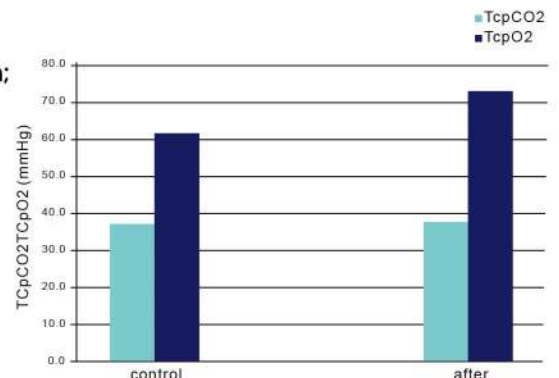

After 15 minutes of nursing, the carbon dioxide content returned to the level before care (blue), while the oxygen content continued to rise

Treatment

The treatment starts with the radio frequency, brings the blood under the skin to the skin surface, and then performs a three-in-one mobilization regimen.

1). RF Radio Frequency

a. Clean face, coated with RF special gel

b. Slowly circulate at a slow speed and focus on a small area until the orange light on the handle lights up.

c. Slowly circling at a slower uniform speed, a small area of a small piece to complete the face

d. Temperature of 37-41°C will allow the skin to comfortably enjoy the entire warm massage process

e. After the completion of the whole face there is a clear increase in tightening effect

2)Oxygenation

a. Clean the face and apply nutrition gel

b. Use dynamic oxygen handles for faster uniform circular motion

c. Chemical reaction through deep cleaning of oxygen heads and gels generates large amounts of carbon dioxide and water, while forming a perfect "weak acidic environment" on the skin surface.

d. Bohr effect occurs on the surface of skin, which automatically generates large amounts of oxygen from inside the skin

e. Oxygen-filled skin cells are arranged more tightly and skin problems are reduced

Oxygenation technology achieves results through two product reactions. One is a disposable treatment head on hand tools, one is a gel of pure natural ingredients; one can be anti-wrinkle regeneration, one can whiten skin. Both products are packaged in separate boxes with 6 doses in each box. Each time with a set of products. The two products have different ingredients and different effects.



Each product portfolio has its own set of boxes:

- Each set consists of 6 individually packaged gels and 6 disposable caps (Capsugens).
- Use one bag of gel and one disposable hand cap (Capsugens) for each treatment
- If the number of care is less than 6 times, the remaining product can be used for daily care.

Two pictures, one is a gel; the other is a one-time treatment head. The combination of the two will produce CO₂ and increase the oxygen content of the skin.

NeeRevive - Anti-wrinkle regeneration

1. Anti-wrinkle regeneration: Lifting firming, nourishing moisture, smooth skin, fade wrinkles.

- Lifting firm
- Nourish and moisten
- Smooth skin
- Fade wrinkles



NeeBringht - Brightening

Whitening & Brightening: Brightens skin, adjusts uneven skin tone, improves pigmentation, and reduces light damage. Ultraviolet light, infrared light, or sunlight can cause damage to the skin, and this product contains an ingredient that can reduce light damage.

- Bright skin
- Adjust uneven skin tone
- Improves pigmentation
- Reduce light damage



RF

Radiofrequency energy can stimulate regeneration of collagen and elastin fibers in the dermis, and can be effective:

- Smooth and firm skin
- Reduces wrinkles and fine lines
- Improve skin texture

4. Use efficiency

Mainly through the specific depth in the skin, the skin cells in the skin and subcutaneous tissues produce high-speed vibration, so that the tissue quickly heated. The increase of temperature promotes the catabolism of subcutaneous fat, stimulates the proliferation and reorganization of collagen and elastic fibers, and achieves the effect of tightening the skin and shaping the face.

- 1) Raise mention tightening cheek skin
- 2) Improve chin lines and reduce puppet patterns
- 3) Improves skin texture, leaving skin delicate and shiny
- 4) Remove neck lines to prevent neck aging
- 5) Improve skin elasticity and contour firmness
- 6) tighten the skin of the forehead and improve the line of eyebrows
- 7) Eliminate wrinkles around the forehead, eyes and lips

5. Instrument advantages

The Oxygeneo device radio frequency technology overcomes two disadvantages of the laser or photon (electric wave skin) acting on the tissue: the penetration depth is not enough, and the barrier of the pigment cells of the epidermis base layer is an ideal skin tissue heating technique.

1. Wrinkle-removing, double effect: Under the skin at a specific depth, the skin cells in the skin and subcutaneous tissue will produce high-speed vibration, so that the tissue will quickly heat up. The increase of temperature promotes the catabolism of subcutaneous fat, stimulates the proliferation and reorganization of collagen and elastic fibers, and achieves the effect of removing wrinkles, tightening the skin and shaping body shape.

2. Non-invasive and painless, safe and comfortable: With ring-shaped electrodes to ensure even skin heat, control RF penetration depth

3. Significant effect, long-lasting effect: Suitable for all parts of the wrinkle, can be used for facial and neck skin wrinkles, face-lift, to double chin. The rejuvenation bubble device radio frequency heats the collagen layer, and the collagen fiber can be contracted. The effect is obvious, and the dermal collagen continues to proliferate after treatment.

4. Treatment is simple, lunch-based treatment: According to the size of the skin to be treated, the treatment process of the rejuvenation bubble radiofrequency needs about 40 minutes. After treatment, normal work can be performed without taking a break.

5. No trace: non-surgical non-invasive skin treatment will not leave any wounds on the skin surface, wrinkle, dissolve fat while tightening the skin, so that the skin becomes more smooth and moist, like a new life





Features: No damage to the skin of plastic technology.

6. Treatment settings

Dynamic oxygen combined with radiofrequency, a course of about 6 times. The first four times a week, the second time a month, can achieve lasting effect.

7. Comparison of instruments

Oxygeneo radio frequency comparison with other radio frequency

Technology	Image	Clinical target	Power	pain Level	Cooling	Visible Clinical Results
Mono Polar		Collagen fibers	High (200-350W)	High (200-350W)	Always	After months
Uni polar (RF radiation)		Collagen fibers	High (200-350W)	High (200-350W)	Always	After months
Bi-Polar		Dermis & hypodermis	Medium (10-200W)	Medium (10-200W)	May be	After days / weeks
Tripollar		Focused energy on dermis & hypodermis	Low (5-50W)	Low (5-50W)	None	Immediate & long term. supported by clinical and histopathological studies

7.indications

1). People with skin wrinkles:

There are crow's feet, forehead lines, nasolabial folds, neck lines, stretch marks on the abdomen, fat lines on the thighs and other wrinkles.

2) People with skin aging, sagging and sagging:

Under the hustle and bustle, there are double chin, sagging eyes, drooping eyebrows and other people;

3) People with loose skin and bloated skin:

The facial skin is loose, and the abdomen and arm have fat.

9. Before and after comparison

1. Improve fine lines around the eyes



The picture shows the contrast between the treatment of eye crow's feet with Oxygeneo. Before the treatment on the left, the crow's feet pattern was obvious. After the treatment on the right, the crow's feet was reduced by about 80%.

2. Improves firmness and smooths skin



Oxygeneo also have excellent healing effects on spur tumors. The following figure shows a young woman. The left picture shows that there are two raised syringomas in her eyes. Through five treatments (right), it can be clearly seen that the syringomas have disappeared, and the skin becomes firm, smooth, and even in color.

3. Bright complexion, lighter spots



Through a contrasting picture of the entire face, you can see some light-colored spots on the circle, including the parts indicated by the arrows on the left picture, and some scattered spots. After about five times of rejuvenating oxygen therapy, the skin's stain, color, and smoothness can be clearly improved.

4. Improve skin texture and brighten skin tone



Oxygeneo can also shrink the pores. The pores on the left side of the figure are much larger than the right side pores. The right side is the effect after five treatments.

10.common problem

① What are the special preparations before and after treatment?

The Oxygeneo device does not require any special preparation, and normal work and life can be performed immediately after treatment. Some people will be short after treatment. The redness of the skin usually disappears within 30 minutes

② What should I pay attention to when treating?

Don't wear any metal products on your body. Do not use cosmetics during treatment.

③ Which part can be treated by the Oxygeneo device?

Cheeks, eyes, crow's feet, forehead, nasolabial fold, neck, chest, upper arm, hand, etc.

④ How does it feel when you have a Oxygeneo?

During treatment, the patient may have a slight burning sensation. The doctor will adjust the energy according to the reflection to obtain a better therapeutic effect.

⑤ How long does it take for each treatment?

According to the size of the treatment site and treatment settings, treatment time varies from 40 minutes to 1 hour

⑥ What can be achieved after treatment?

Due to skin collagen regeneration after treatment, skin elasticity can be restored, skin becomes delicate, elastic, skin wrinkles and problems The skin also improves.

⑦ How long can you see the treatment effect?

Immediately after treatment, the elasticity of the skin can be restored. Due to the gradual regeneration of collagen in the dermal layer of the skin, the muscles within 2-3 days There will be further improvement in the skin

⑧ Can the revival bubble device replace the surgical scalp?

Dynamic oxygen and radiofrequency are the most effective wrinkle removing methods currently used for non-surgical treatment. For the skin to start with mild to moderate sagging, small wrinkles appear. Has obvious advantages.

⑨ What is not suitable for RF wrinkle removal?

Patients with heart disease, who are equipped with cardiac pacemakers, pregnant women, and who have been injected with liquid silicone in the treatment area within 3 months are not suitable Do Oxygeneo

⑩ How long can the effect of the Oxygeneo device last?

The sustained effect of the revitalizing bubble device is related to the degree of aging of the individual's physique. The research report points out that satisfactory results can be achieved at least To maintain more than 1 year or even longer. The duration of maintenance may also vary according to the individual's natural skin aging rate.